



# HEARTCENTRIX® Recommended Requirements

Computer and accessory requirements necessary to operate HeartCentrix software

Recommended server requirements	Windows 2003 Server	Windows XP® Professional
<b>Number of users</b>	Unlimited	3 concurrent
<b>Computer</b>	•	•
2.0 GHz Processor or better	•	•
256KB Cache	•	•
RAM	2.0 GB	1.0 GB
80 GB Free Hard Disk Space (Application and Data Storage)	•	•
Network Interface Card	•	•
CD ROM Drive, 32X	•	•
<b>Video Card:</b> must support 1024 X 768 at 65K Colors or 1280 x 1024 at 65K Colors.	•	•
<b>Mouse</b>	•	•
<b>Keyboard</b>	•	•
<b>Operating System</b>	Windows 2003 Server R2 (Standard Edition) (32-bit version)	Windows XP Professional SP2 (32-bit version)
<b>Monitor</b>	•	•
<b>Internet Browser Compatibility</b>	•	•
Microsoft® Internet Explorer®	Version 6, 7, or 8	Version 6, 7, or 8
Microsoft Internet Information Services (IIS)	Version 6.0 required	Version 5.0 required

## Requirements

1. Uninterruptible power supply. Provides the ability for a graceful system shutdown in the event of a power failure.
2. Off-line backup of database. Backup device capable of backing up stored HeartCentrix Data. Off-line nightly backup of the database is the customer's responsibility.
3. Remote support access to this system. This access can be via a RAS connection directly to this PC, through a dial-up RAS Server, VPN, WebEx™, or similar internet access application (e.g. GoToAssist application). If dial-up connection is directly to this machine a modem and free communication port are required.

## Other considerations

- + System performance on customer supplied computers is the responsibility of the customer.
- + Anti-Virus scanning software should be installed and is the responsibility of the customer.
- + The computer cannot have had any version of DB2 except 7.2 installed prior to the HeartCentrix installation. Uninstalling DB2 is not enough for a clean installation of DB2 version 7.2.
- + The computer can not be a domain controller.

## HeartCentrix Client Workstation Supported Operating Systems

- + Windows XP Professional SP2 (32- and 64-bit version)
- + Windows 7 (32- and 64-bit version)
- + Windows VISTA (Business, Ultimate, Premium, Home) (32- and 64-bit version)
- + Windows 2000 Professional (32- and 64-bit version)

**Cardiac Science Corporation** • 3303 Monte Villa Parkway, Bothell, WA 98021 USA • 425.402.2000 • US toll-free 800.426.0337 • Fax: 425.402.2001 • info@cardiacscience.com

**Orders and Customer Care (US and International)** • Fax: 425.402.2001 • care@cardiacscience.com

**Technical Support** • (US) Fax: 425.402.2022 • technicalsupport@cardiacscience.com • http://websupport.cardiacscience.com/webchat/ • (International) internationalsupport@cardiacscience.com

**Cardiac Science International A/S** • Kirke Vaerloesevej 14, DK-3500 Vaerloese, Denmark • +45.4438.0500 • Fax: +45.4438.0501 • international@cardiacscience.com

**United Kingdom** • The Manse, 39 Northenden Road, Sale, Manchester, M33 2DH, United Kingdom • +44.161.926.0000 • uk@cardiacscience.com

**France** • Parc de la Duranne, 565 Rue René Descartes, 13857 Aix en Provence cedex 3, France • +33.4.88.19.92.92 • france@cardiacscience.com

**Central Europe (D, A, CH)** • Oskar-Schindler-Strasse 3, D-50769 Köln, Germany • +49.2132.93.5750 • centraleurope@cardiacscience.com

**China** • 6/F South Building, 829, Yi Shan Road, Shanghai 200233, China • +86.21.6495.9121 • china@cardiacscience.com



Cardiac Science, the Shielded Heart logo and HeartCentrix are trademarks of Cardiac Science Corporation. All other trademarks are property of their respective owners. Copyright © 2009 Cardiac Science Corporation. All Rights Reserved. 70-00315-01 rC  
Authorized EU representative • MDSS GmbH, Schiffgraben 41, D-30175 Hannover, Germany

